max 400 mm (15½")

max 75 kg (165 lb)


## Manatee User Guide


EN 12182  
ISO 17966

2019-11 rev.06

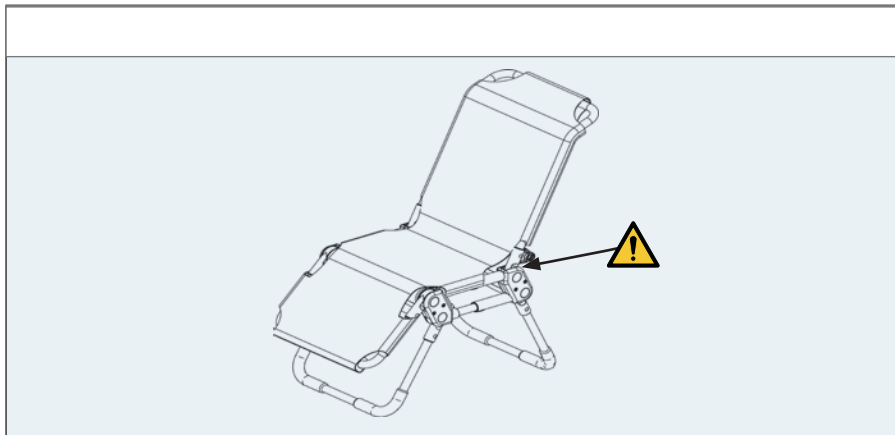
	3	ENGLISH DEUTSCH	19 - 22 23 - 26
	4	NEDERLANDS DANSK	27 - 30 31 - 34
	5	NORSK SVENSKA	35 - 38 39 - 42
	6	SUOMI ESPAÑOL	43 - 46 47 - 50
	7	FRANÇAIS ITALIANO	51 - 54 55 - 58
	8	PORTUGUÊS PORTUGUÊS DO BRASIL	59 - 62 63 - 66
	9	РОССИЯ POLSKI	67 - 70 71 - 74
	10	ČESKY 中國	75 - 78 79 - 82
	11	日本 ΕΛΛΗΝΙΚΗ	83 - 86 87 - 90
	11		
	12		
	13 - 15		
mm (inch)	16 - 17		
	18		


**MANATEE 0**

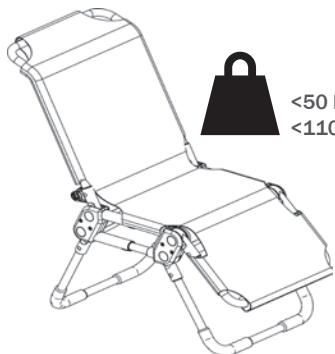
**MANATEE1**












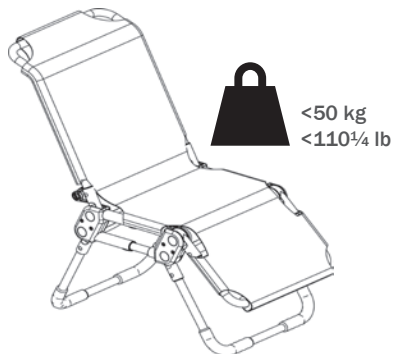
**MANATEE 2**

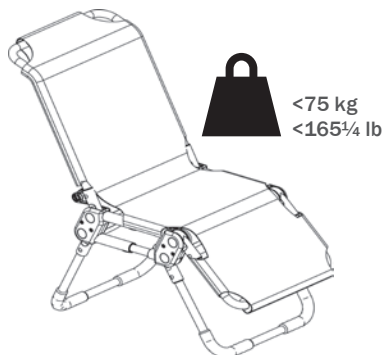**MANATEE 3**

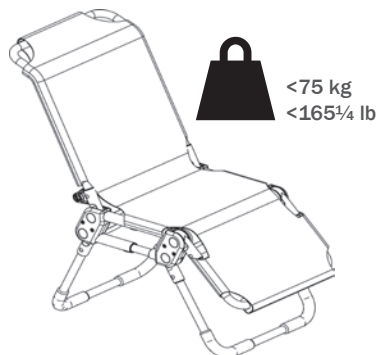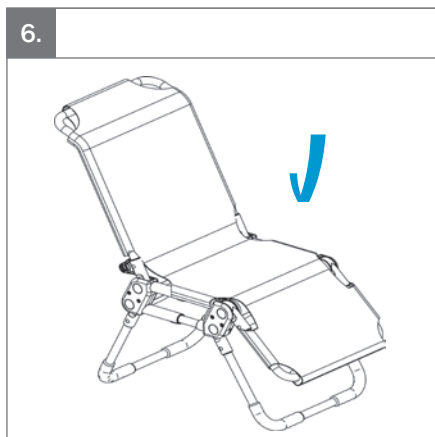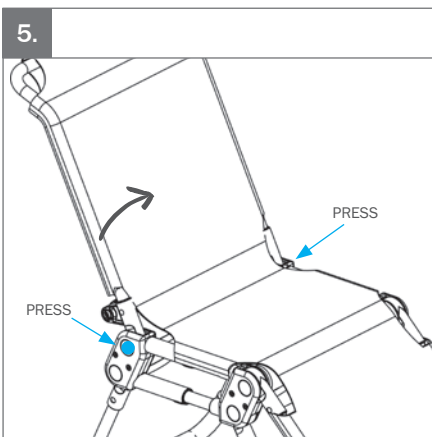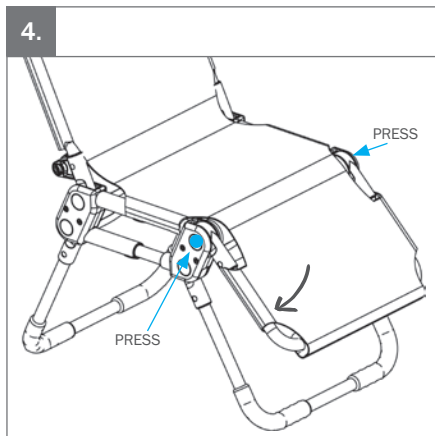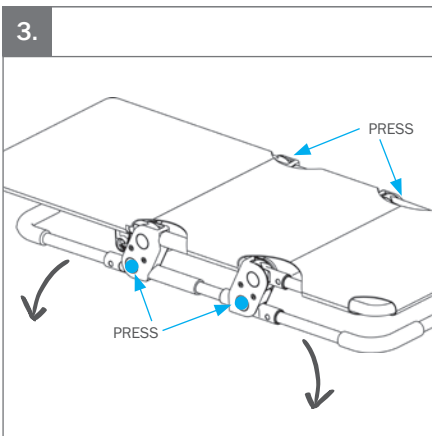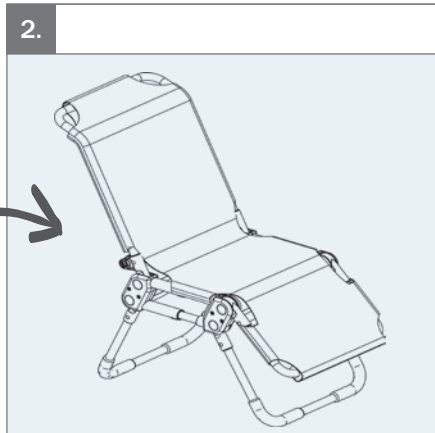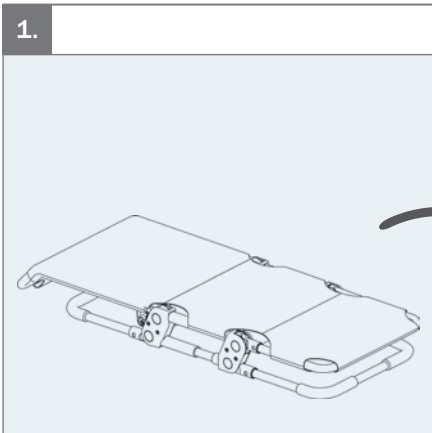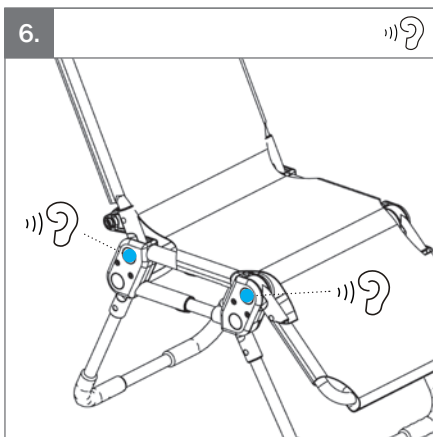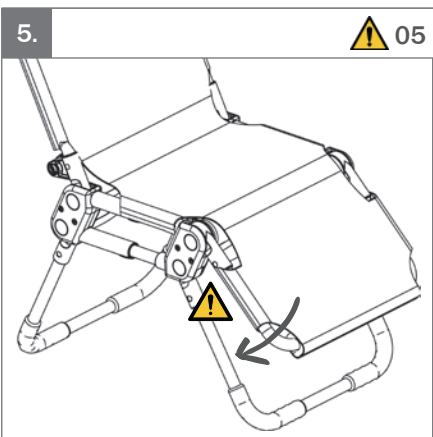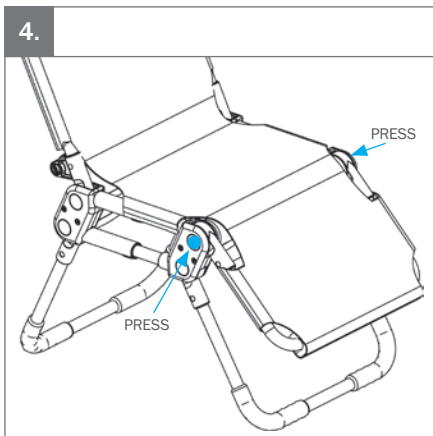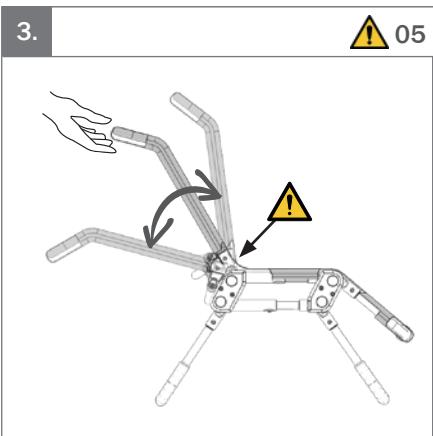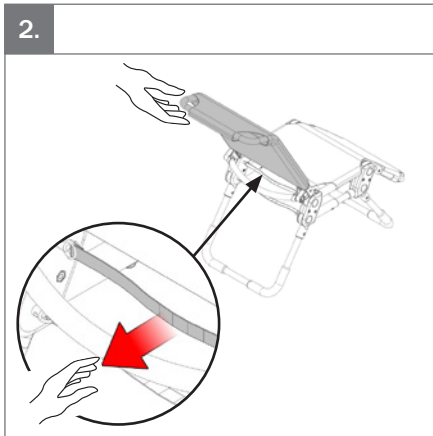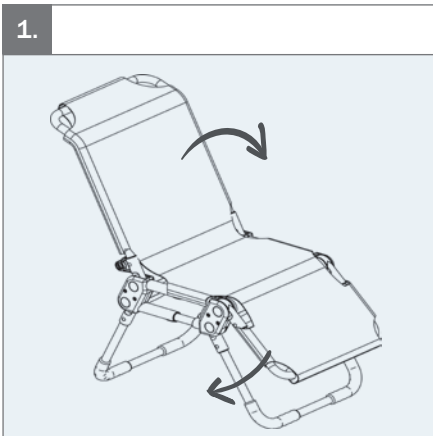
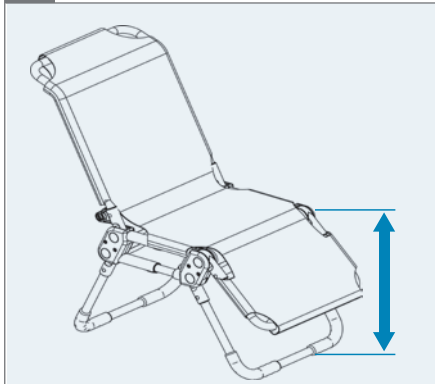
1.

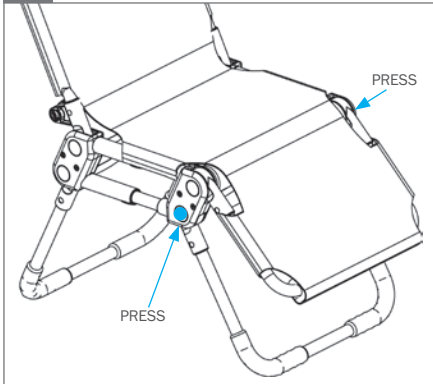
2.


3.

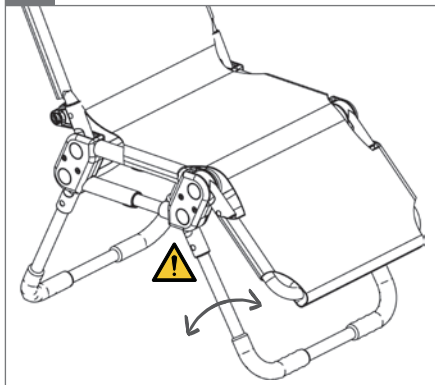05

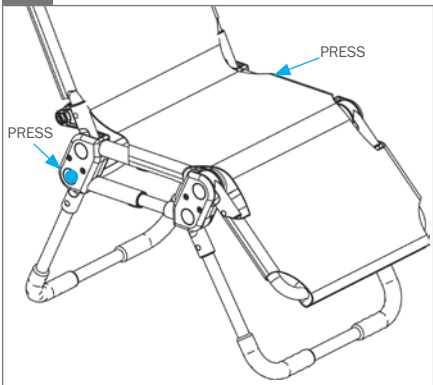
4.


5.

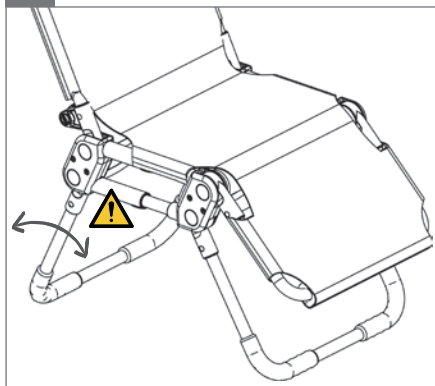05

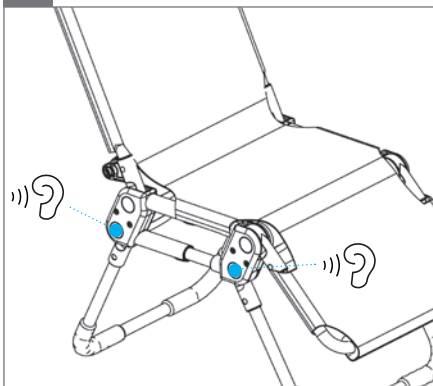
6.


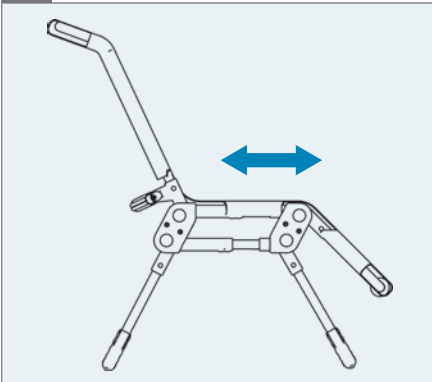
1.

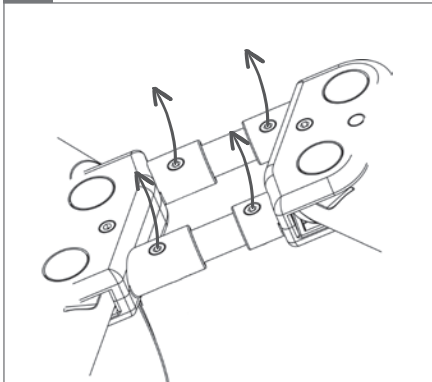
2. Mantee 0 -  $\varnothing$  4 mm

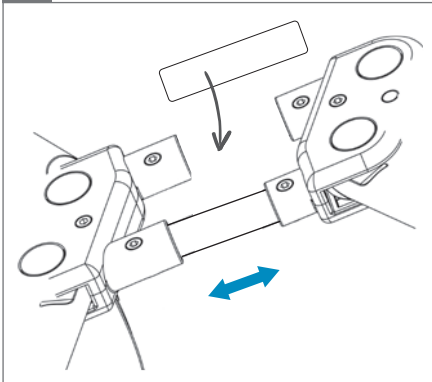
3. Mantee 0

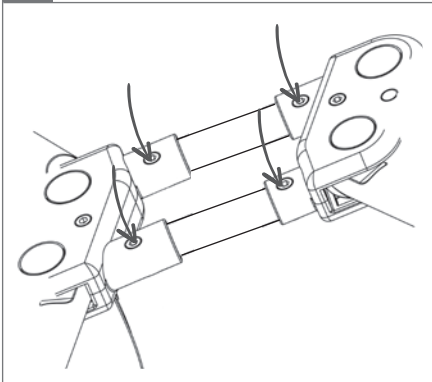
4. Mantee 0 -  $\varnothing$  4 mm, 2,5 Nm

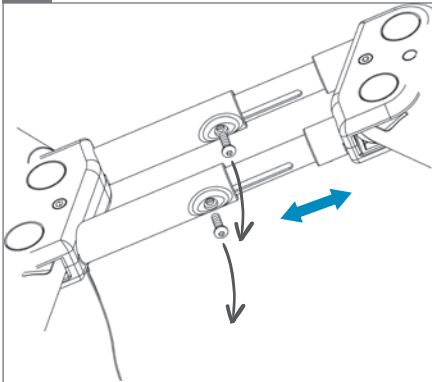5. Manatee 1+2 -  $\varnothing$  4 mm

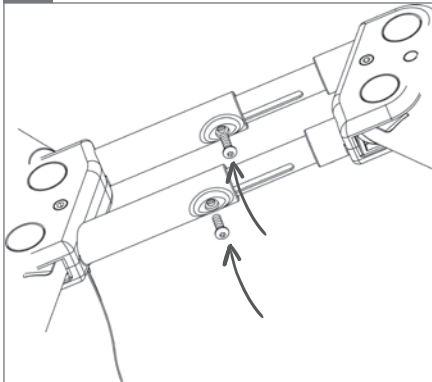
6. Manatee 1+2 -  $\varnothing$  4 mm, 2,5 Nm


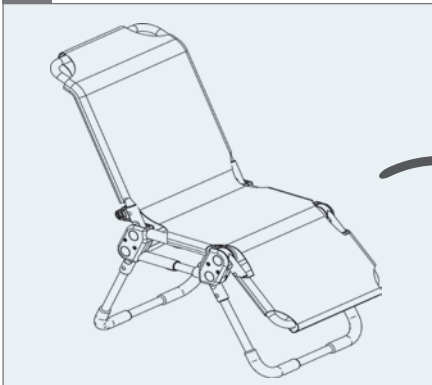
1.

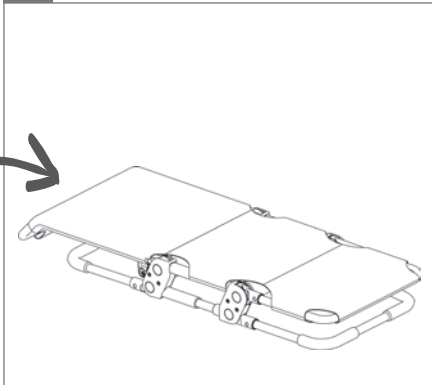
2.

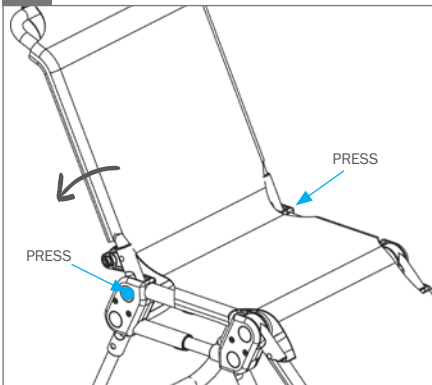
3.

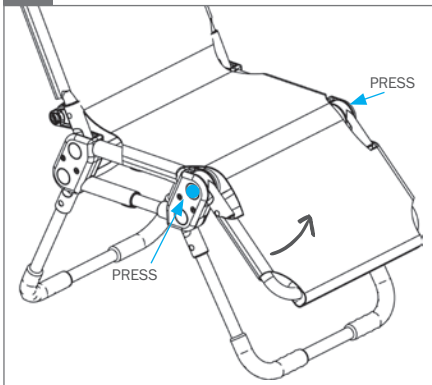
4.




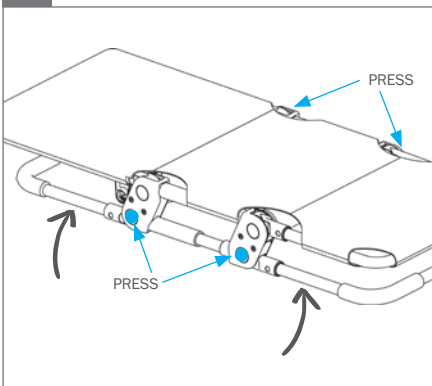
5.

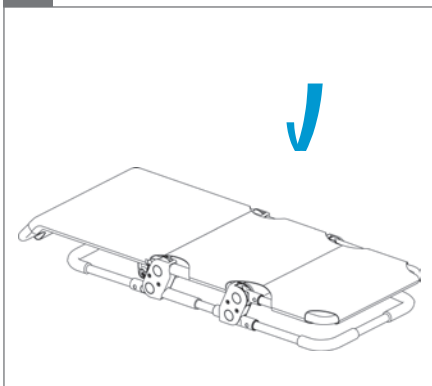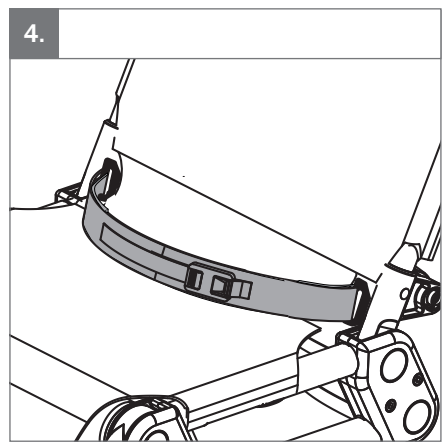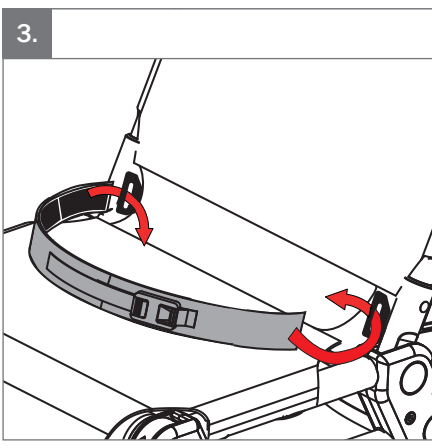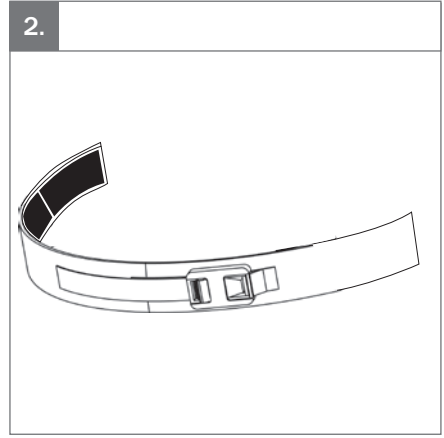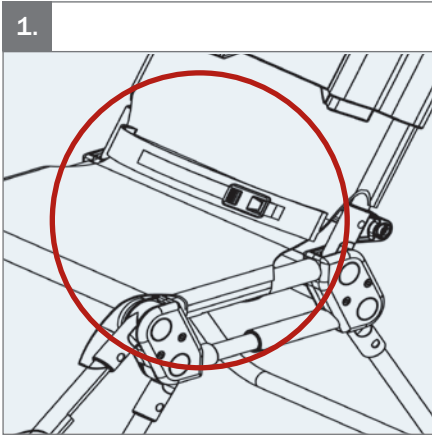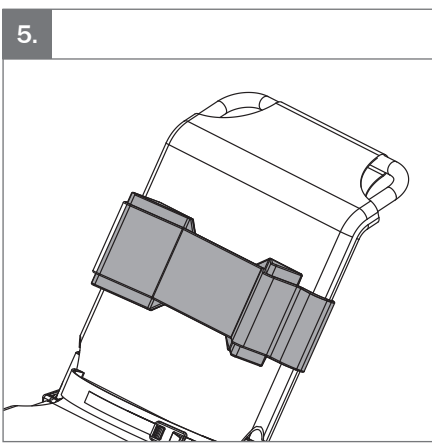
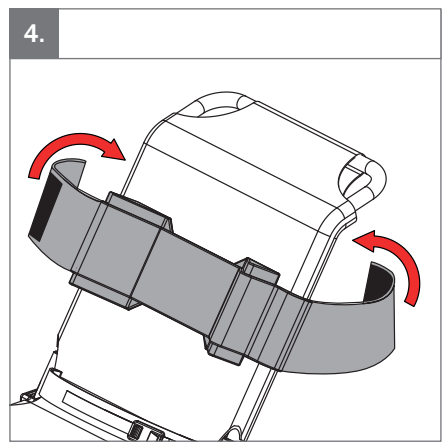
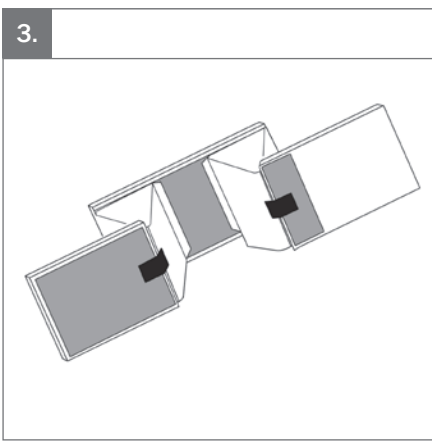
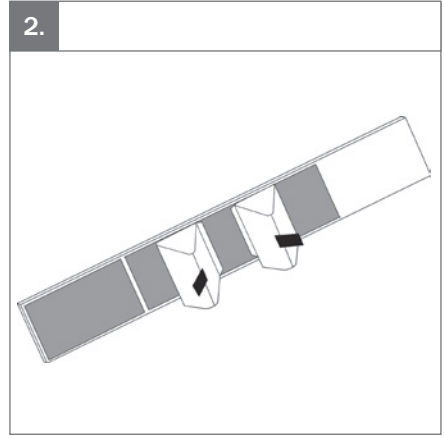
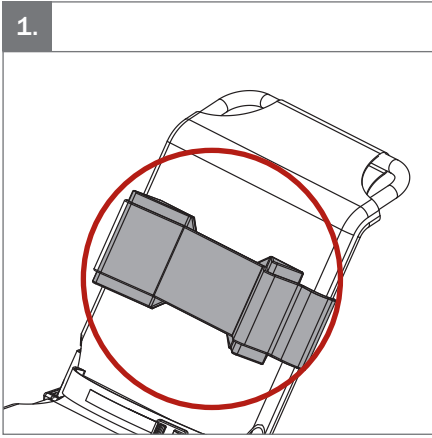
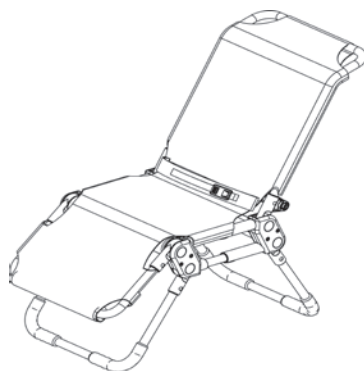
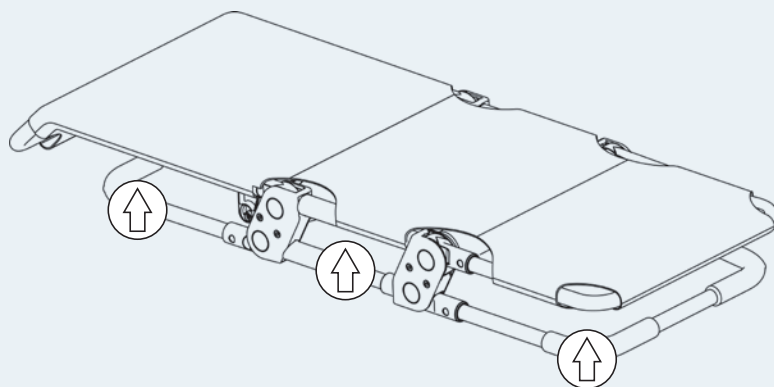
6.





Size 0

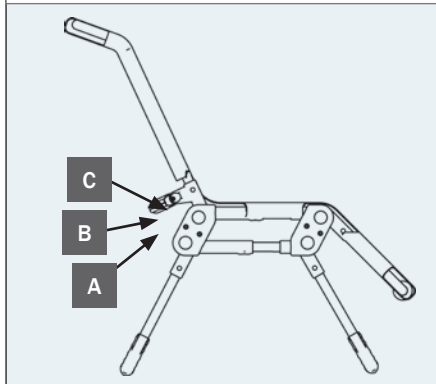Size 1 - 3

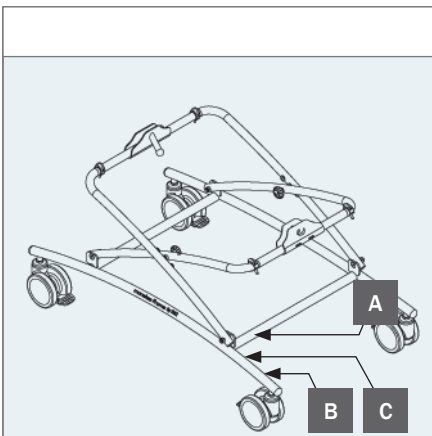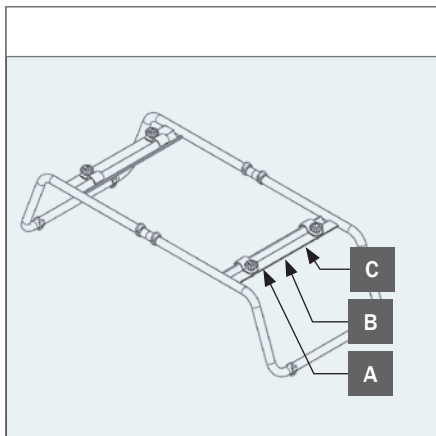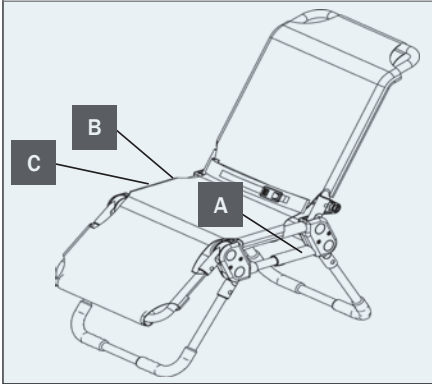A

Max load: xxx kg  
 DMR ver.:XXXX  
 Art. no.:XXXXXXX  
 Product:XXXX - Size X

B

**R82**  
 Parallelvej 3  
 DK-8751 Gedved  
 R82.org  
 Made in Malaysia YYY-MM-DD

C

**EN:** For indoor use**NO:** For innendørs bruk**FR:** Utilisation intérieure**RU:** Для использования в помещении**JP:** 屋内使用**DE:** Für den Innengebrauch**SV:** För inomhusbruk**IT:** Per uso interno**PL:** Do użytku wewnątrz**GR:** Κατάλληλο για εσωτερική χρήση**NL:** Voor binnenhuis gebruik**FIN:** Sisäkäyttöön**PT:** Para utilização em interiores**CZ:** Pro vnitřní používání**DK:** Til indendørs brug**ES:** Para uso en interiores**BR:** Para uso interno**SN:** 适用于室内使用

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


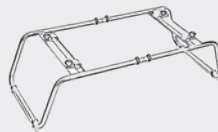
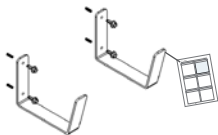
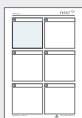
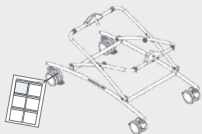
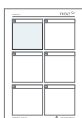
9996097456
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
## ENGLISH

We appreciate your choice of a new product from R82 - a worldwide supplier of technical aids and appliances for children and teenagers with special needs. To fully benefit from the options offered by this product please read this User Guide before use and save it for future reference.

### Intended purpose

The Manatee is a bathing chair suitable for disabled children. The chair is designed to place the user in exactly the right sitting position. The depth of the seat can be adjusted, allowing the chair to grow with the child. The product is for indoor use in a bathroom environment.

The target group for this device is disabled children with cerebral palsy (CP). Also, children with muscular dystrophy, spina bifida, spinal cord injuries and neurological impairments can be subject to the device. The product is suitable for children with GMFCS level 3-5.

### Declaration of Conformity

This product conforms to the requirements of the Medical Devices Directive (93/42/EEC). The CE mark must be removed when rebuilding the product, when used in combination with a product from another manufacturer or when using other than original R82 spare parts and fittings. Further, this product conforms to the requirements according to:

- EN 12182
- ISO 17966

### Disposal

When a product reaches the end of its lifetime, it should be separated by material type so that the parts can be recycled or disposed of properly. If required, please contact your local dealer for a precise description of each material. Please contact your local authorities to be informed of the options available for recycling in an environmentally-friendly way.

### Accessories and spare parts

The products from R82 can be supplied with a variety of accessories which comply with the needs of the individual user. Spare parts can be ordered on request. Find the specific accessories on our website or contact your local dealer for further information.

### R82 Warranty

R82 offers a 2-year warranty against defects in workmanship and materials and a 5-year warranty on breakage of the metal frame caused by defects in welds. The warranty would be adversely affected if the customer's responsibility of servicing and/or maintenance is not carried out according to the guidelines and intervals prescribed by the supplier and/or stated in the User Guide. For further information, we refer to the R82 homepage/download.

The warranty is only granted in the country where the product was purchased and if the product can be identified by the serial number. The warranty does not cover accidental damage, including damage caused by misuse or neglect. The warranty does not extend to consumable items e.g. tires or upholstery, which are subject to normal wear and tear and need periodic replacement.

The warranty is null and void if other than original parts/accessories are used, or if the product has not been maintained, repaired or altered according to the guidelines and intervals prescribed by the supplier and/or stated in the User Guide. R82 reserves the right to inspect the product being claimed for and the relevant documentation before agreeing to the warranty claim, and to decide upon whether to replace or repair the defective product. It is the customer's responsibility to return the item being claimed for under warranty to the address of purchase.

### Presales information

- [support.r82.org](http://support.r82.org)

## Safety

Signs, symbols and instructions placed on this product must never be covered or removed and must remain present and clearly legible throughout the entire lifetime of the product. Immediately replace or repair illegible or damaged signs, symbols and instructions. Contact your local dealer for instructions.

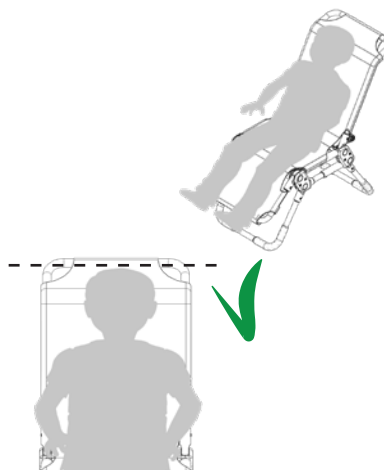
In case of an adverse event occurred in relation to the device, incidents should be reported to your local dealer and the national competent authority in a timely manner. The local dealer will forward information to manufacturer.

## Caregiver

- Read the "User Guide" carefully before use and keep it for future reference. Incorrect use of this product may cause serious injury to the user
- Always use the correct lifting techniques and aids for this purpose
- Never leave the user unattended in this product. Ensure permanent supervision by an adult
- Repair/replacements must only be made using new original R82 spare parts and fittings and carried out in accordance with the guidelines and service intervals prescribed by the supplier
- If there is any doubt as to the continued safe use of your R82 product or if any parts should fail, stop using the product immediately and contact your local dealer as soon as possible

## Environment

- Observe the surface temperature on the product, before placing an user in it. This goes especially for users with insensitive skin as they cannot feel the heat. If the surface temperature is above 41 °C then allow the product to cool before use

## User

- If the user is close to the max. load limit and/or has a lot of involuntary movements, e.g. rocking, a larger size with a higher max. load or a different R82 product should be considered
- The head of a user must always be within the top of the frame

## Product

- Carry out all positional adjustments on the product and accessories and ensure that all knobs, screws and buckles are securely fastened before use. Keep all tools out of the reach of children
- Ensure the stability of the product is maintained before placing the user in it
- Please check on a regular basis that the zippers are safely closed.
- Check that all release buttons mesh into place after adjustment
- Ensure the Anti-slip rubber covers are functional
- Check on the cover, belts and accessories and make sure that the Velcro fastens securely
- The enclosed belts and accessories cannot be used as long-term supports but may only be used as an extra help when positioning the user.

## Service information

Maintenance of a medical device is wholly the responsibility of the owner of that device. Failure to maintain a device in accordance with the instructions may invalidate the device's warranty. Furthermore, failure to maintain a device may compromise the clinical condition or safety of users and/or their caregivers. Do not perform service and maintenance while the user is in the product.

If the product is subject to tap water containing chlorine, then discoloration of plastics and fabric can be expected.

### 1st Day

- Read the User Guide carefully before use and store it in a safe place for future reference

### After every use

- Rinse the product with water to remove leftovers from soap and organic material

### Daily

- Clean the anti-slip rubber covers, using a cloth with warm water and mild detergent and wipe them dry
- Visual inspection. Repair or replace any damaged or worn-out parts
- Check all fittings and buckles for signs of worn out parts
- Ensure that all fasteners are in place and fitted correctly
- Be aware of any MAX mark indications before adjustment

### Monthly

- All nuts and bolts on this product must be checked and tightened to avoid unnecessary failures

- Oil the swing-away parts. We recommend you use a professional lubricant system

#### Every 6 months

- Inspect the frame for cracks or signs for worn-out parts and carry out annual service. Never use a product which appears to be substandard or faulty

#### Disinfection

The product can be disinfected with a 70% disinfectant IPA solution. It is recommended to wipe off any residue and dirt from the product, using a cloth with warm water and a mild detergent/soap without chlorine and let it dry before disinfection.

Before disinfection

Remove the following items and clean them separately: Cover, Cushioning & Accessories

#### Washing

The cover can be washed at max. 50 °C using a mild detergent in a washing machine.

#### Washing machine

This product can be washed at 60° using a mild detergent for 10 minutes in a washing machine designed for medical devices. Use the machine's drying function to dry the product.

This product can be washed at 85° using a mild detergent for 3 minutes in a washing machine designed for medical devices. Use the machine's drying function to dry the product.

#### Before washing

Remove the following items and clean them separately: Cover, Cushioning & Accessories

#### Service interval

A detailed inspection of the product must be made every 12 months (every 6 months for heavy users) and every time the product is reissued for use. The inspection should be carried out by a technically competent person who has been trained in the use of the product.

The lifetime of this product in normal use is 5 years if all maintenance and servicing is carried out in accordance with the manufacturer's instructions and demonstrably recorded.

#### Service Check List:

Check and adjust the following functions as per their intended use. Repair or replace any damaged or worn-out parts. Please check the:

- Backrest and leg support is adjustable and can be locked in position
- Zipper(s) & Velcro
- Chair is foldable/unfoldable
- Nuts and bolts on the product are tightened
- Product for cracks or signs for worn-out parts
- All labels on the product are intact

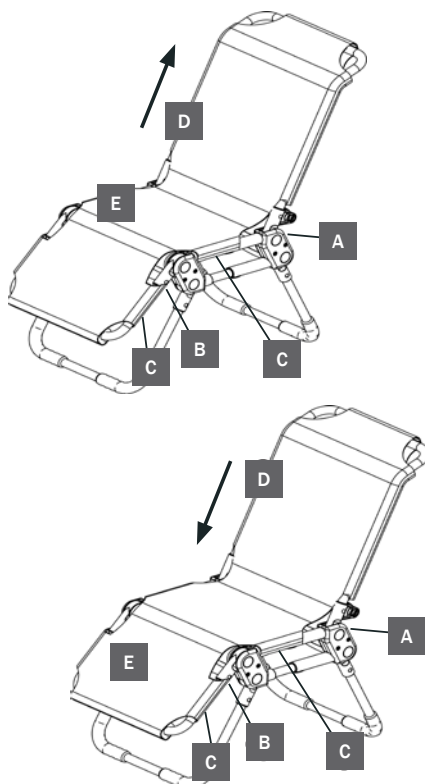
#### Dismounting and Mounting the Cover

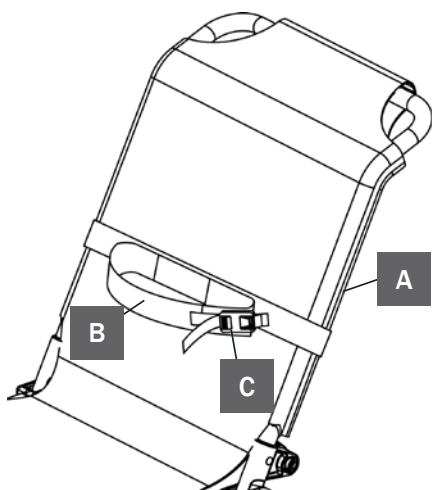
Follow the description below when the cover needs to be dismounted and washed:

1. Unzip the cover on the back (A) and underneath the seat (B)
2. Open the velcro underneath the chair (C) and the cover can be pulled up (D) and off the chair

Follow the description below to remount the cover on the chair:

1. Put the cover over the back and pull down (D). Make sure that the foam cushions in the cover are placed around the frame
2. Zip the cover on the back of the chair (A)
3. Mount the small cover piece with the zipper underneath the chair (B). Make sure that the velcro is pointing downwards
4. Tip up the leg support (E) and the velcro underneath the seat (C) and underneath the leg support (C) can be closed. Make sure that the foam cushions in the cover are placed around the frame
5. Tip down the leg support (E) and make sure the velcro is fastened correctly

## Accessories

### Body support

Please follow the description below to mount the body support the correct way:

1. Fasten the strap with velcro around the back (A)
2. Place the user in the chair. Mount the strap (B) around the body
3. Tighten the body support with velcro and the buckle (C)

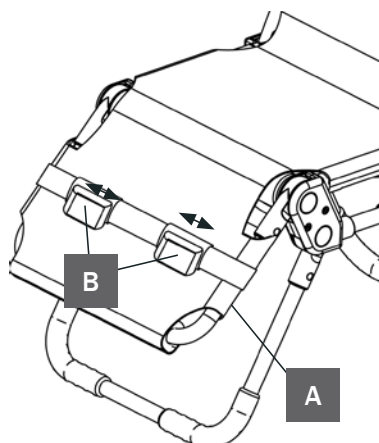
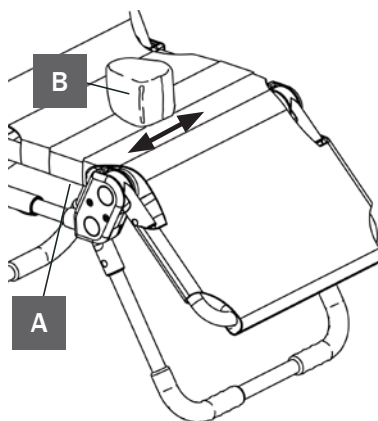
The body support may not be used as a long-term support. This support is only designed to make it easier to place the user in the chair.

### Pommel with strap

Please follow the instructions below to mount the pommel and the strap the correct way:

1. Fasten the strap with velcro underneath the seat (A)
2. Adjust the pommel (B) sideways to gain the optimal position

The pommel with strap may not be used as a long-term support. This support is only designed to make it easier to place the user in the chair.

### Straps for leg control

Please follow the description below to mount the straps the correct way:

1. Fasten the strap with velcro underneath the leg support (A)
2. Adjust the straps (B) sideways to gain the optimal position. Fasten with velcro

The straps may not be used as a long-term support. This support is only designed to make it easier to place the user in the chair.