
# Recondition instruction **molift®**

## Molift Transfer Pro

by Etac

RE70199 - 2025-04-04 - Rev. 4.0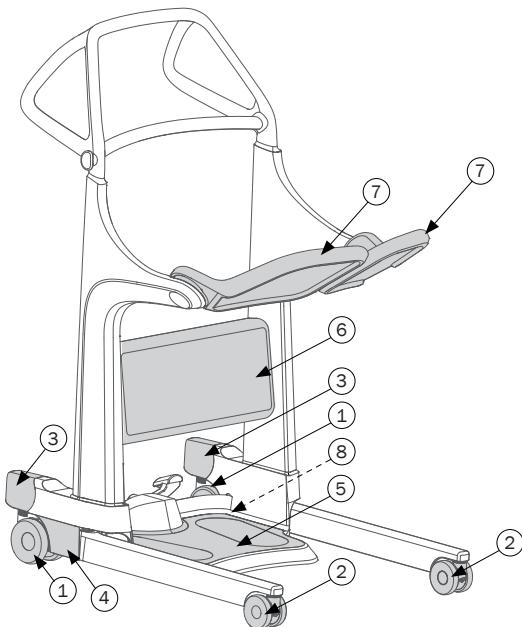

### Reconditioning illustration      Page

1. Rear castors	2
2. Front castors	2
3. Brake pedal	2
4. Leg adjustment covers - LH/RH	2
5. Footplate	3
6. Leg support	3
7. Seat pad	3
8. Leg adjustment mechanism	4

### Reconditioning language      Page

English	5
Svenska	7
Norsk	9
Dansk	11
Suomi	13
Deutsch	15
Nederlands	17
Français	19
Italiano	21
Español	23





### Cleaning      Page

English	6
Svenska	8
Norsk	10
Dansk	12
Suomi	14
Deutsch	16
Nederlands	18
Français	20
Italiano	22
Español	24


www.etac.com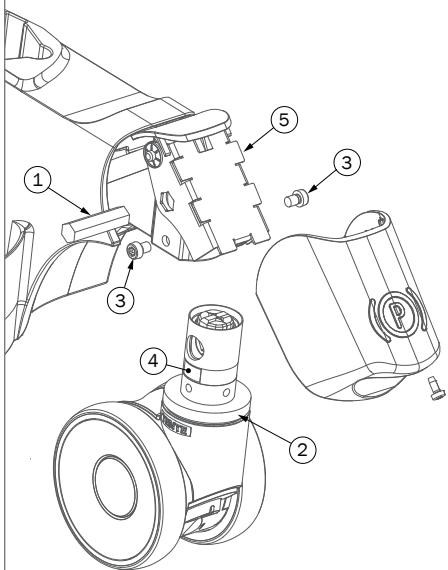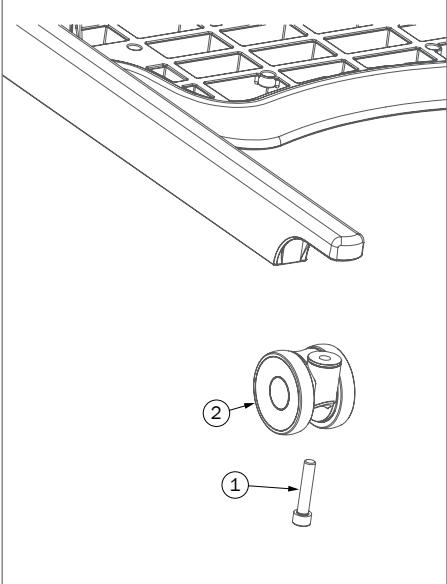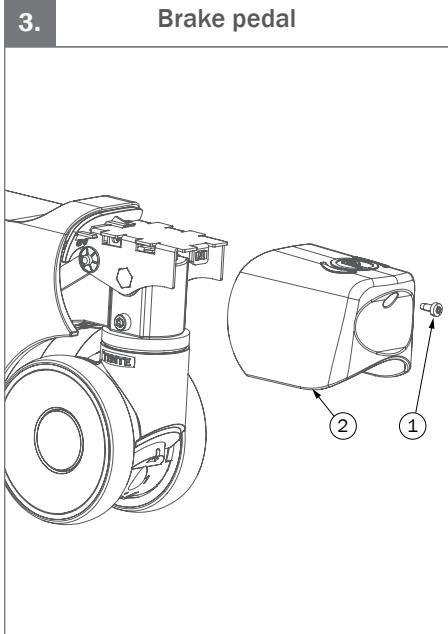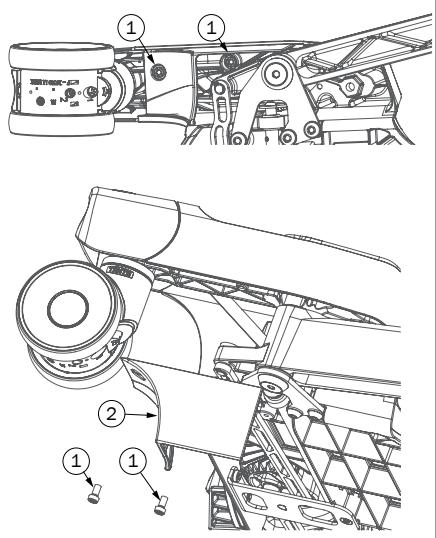

**1.****Rear castors****2.****Front castors****3.****Brake pedal****4.****Leg adjustment covers – LH/RH**


**5.**

### Footplate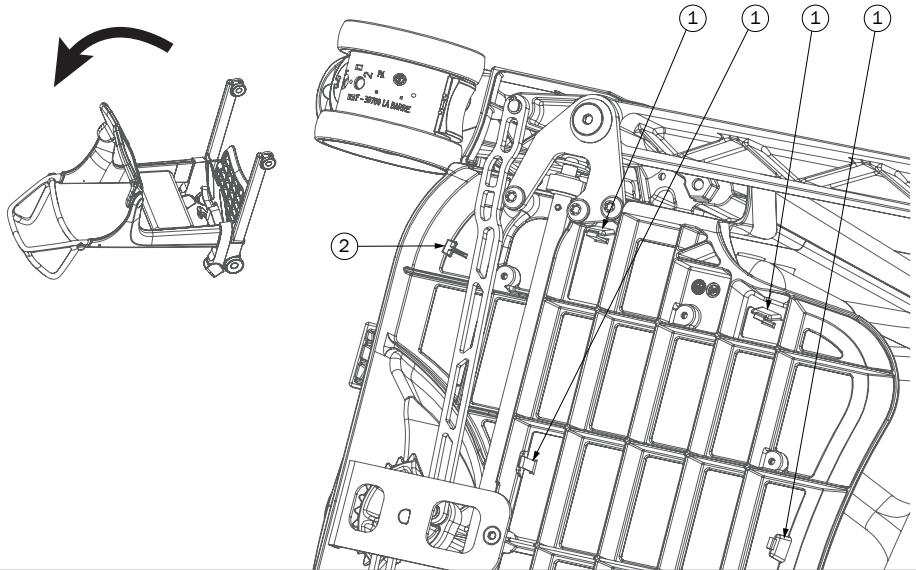


**6.**

### Leg support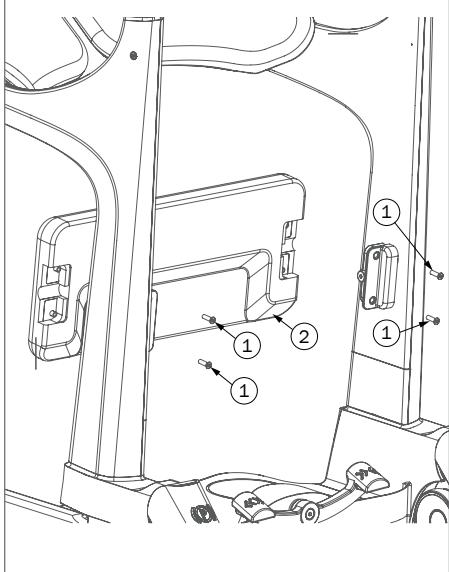





**7.**

### Seat pad LH/RH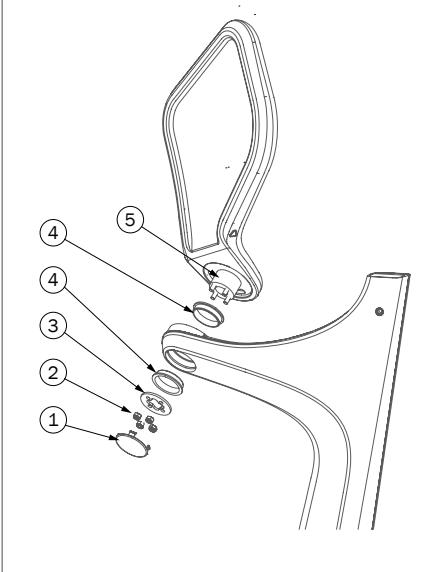

8.

## Leg adjustment mechanism

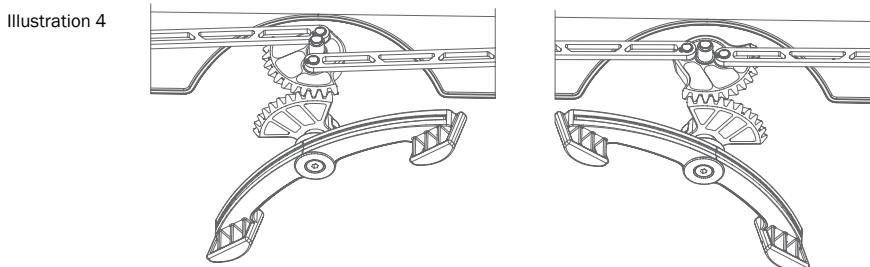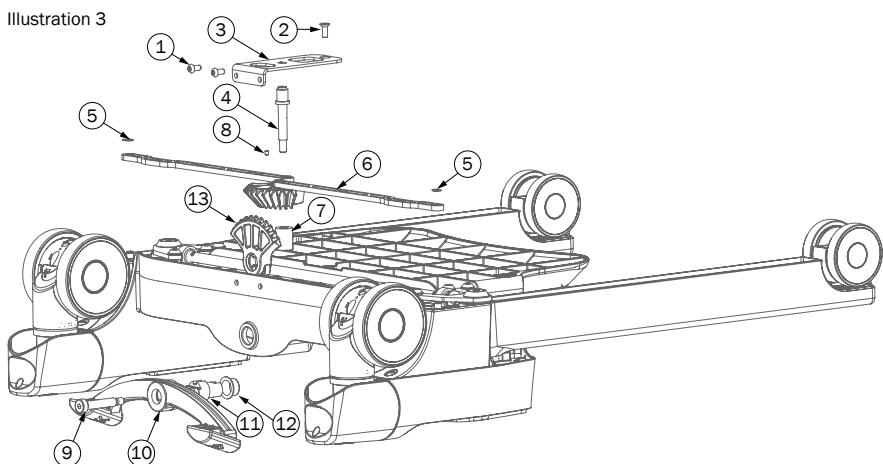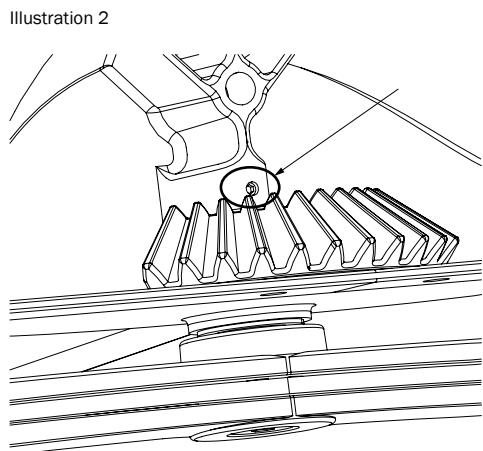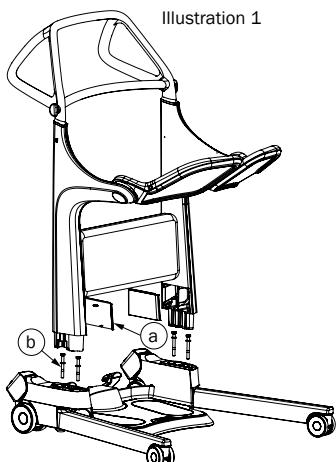**1. Rear castors**

- Place device on a platform with the rear castors over the edge so they are free to remove
- Remove brake pedal. See "Brake pedal"
- Push down to activate the brake pedal
- Remove the hexagon shaft (1)
- Support castor (2) (castor will fall out) and unscrew both bolts (3) (use TX30)
- Insert the hexagon shaft into the castor and activate the brake (use 11mm wrench)
- Insert new castor into the base. Yellow Label (4) on castor shaft must be on the left
- Install the two new bolts
- Install the hexagon shaft (brake must be activated when doing this)
- Install the brake pedal
- Check that the brake works. Must be able to be activated/deactivated from both brake pedals and verify that both castors lock

**2. Front castors**

- Place device on a platform with the front castors over the edge so they are free to remove
- Remove bolt (1) (use hex 8)
- Castor (2) is free to remove
- Install new castor w. new bolt

**3. Brake pedal**

- Unscrew screw (1) (use TX20)
- Pull hard back on brake pedal (2) to remove
- Slide new brake pedal into place
- Install new screw
- Check that the brakes works

**4. Leg adjustment covers – LH/RH**

- Turn castor as shown and move legs to outmost position to access bolts (1)
- Unscrew the 2 bolts
- Remove cover (2)
- Install new cover w. 2 new bolts

**5. Footplate**

- Activate brakes on device
- Pull the device back and lay it on the handle so that the bottom of the lifter is free
- Flip out all 8 front hooks on the footplate. Start from front to back. Footplate will be free to remove
- Install new/cleaned footplate by placing rear hooks in position and rotate footplate into position. Press footplate into position
- Check that the rear hooks are correctly placed (no gap under the footplate at the rear)

**6. Leg support**

- Unscrew the 4 bolts (1) (use TX20)
- Remove leg support (2)
- Install new leg support. Note: Cutout in leg support must face down as shown
- Install new bolts

**7. Seat pad – Same procedure LH/RH**

- Remove red cover (1) (use small screwdriver)
- Unscrew the 4 nuts (2) (use 10mm socket)
- Remove the washer under nuts (3)
- Pull out seat pad (5)
- Remove the two bushings (4)
- Install two new bushings into the frame
- Install seat pad into the bushings. Note: Bushings must stay in place during installing seat pad, and the slot in the seat pad must align with the stop pin
- Install new washer and 4 new nuts
- Check that the seat pad rotates freely from stop to stop
- Install new red cover

## 8. Leg adjustment mechanism

- Activate brakes on device
- Remove the top part of the hoist as one part (handle, side frames and leg support) by removing the two small bottom covers inside the side frames (a) (use small screwdriver) and then un-tighten and remove the four bolts and washers (b) (use 13mm wrench) (see ill. 1)
- Pull out the assembled top part of the device and set aside
- Place the base upside down on a secure surface with access to the leg adjustment mechanism
- Remove bolts (1 & 2) (use TX30)
- Take off bracket (3) and shaft (4)
- Remove lock rings (5) (use proper lock ring tool)
- Remove leg adjustment rod assembly (6) and spacer (7)
- Loosen setscrew (8) (use hex 2) (see ill. 2)
- Unscrew bolt for pedal (9) (use hex 5) and remove pedal (10) and shaft (11)
- Remove bushing (12). Pry out using screwdriver
- Take out gear wheel (13)
- Insert new gear wheel (13)
- Install bushing (12). Use shaft (11) as guide

 Ensure gear wheel align with bushing and can rotate freely

- Install Shaft (11) and pedal (10) and tighten bolt (9) (use hex 5)

 Ensure pedal can rotate freely

- Tighten setscrew (8) (use hex 2)
- Install gear pull bar assembly (6) and spacer (7)

 Ensure gear wheel are aligned with the other gear wheel as shown (see ill. 4)

- Install new supplied lock rings (5) (use proper lock ring tool)
- Place bracket (3) and shaft (4) on gear pull bar assembly (6)
- Install and tighten bolts (1 & 2) to 10Nm (use TX30)

 Check that the leg adjustment mechanism runs freely from side to side

- Place the assembled base on the floor, activate the brakes, and position the assembled top part of the device onto the base
- Install the 4 bolts and 4 washers (b) and tighten bolts to 10Nm
- Install the two bottom covers (a)

Checklist after assembly - Make sure that:

- There are no unused screws, washers or covers
- The device appears stable and safe
- Legs move in/out in smooth movement when leg adjustment is activated
- The leg support easily tilts up/down
- All wheels rotate and turn smoothly

## Cleaning

Clean on a regular basis. Clean surfaces with a damp cloth using an appropriate pH-neutral detergent. Do not use solvents or strong liquids as this may damage surfaces on the device. For disinfection when needed; use iso-Propyl alcohol or similar. Avoid abrasive cleaning products.

 Remove hair and filth from the wheels to ensure the wheels rotate freely and the brake functions properly.

 Make sure not to damage or remove labels when cleaning.

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**Etac Ltd.**
**Etac A/S**